Micromanipulation Pipettes

US

EN: Indication for use

Intended use – see label.

Product Description

Micromanipulation Pipettes* are sterile, pyrogen free, disposable, borosilicate glass lab products designed for manipulation of cells in vitro.

*Blastomere Biopsy Pipette, Denudation Pipette, Hatching Pipette, Holding Pipette, ICSI Pipette, PZD Pipette, Transfer Pipette

Storage Instructions

Store at +8 to 25° C.

Directions for use

Check the packaging for signs of any damage.

Remove the foil seal by pulling it over the silicon holder. Twist the silicone holder, without extracting it, to release it from the glass tube. Then carefully remove the silicone holder by gently drawing it out of the glass tube being careful not to touch the pipette on the side of the packaging.

Detach the pipette from the silicone holder by gently bending the silicone holder. This will release the back, or proximal end, of the pipette from the holder. Grasp the released end and carefully remove the pipette completely from the silicone holder.

This product must be used under the guidance of personnel who are well qualified in the procedures involving these pipettes.

Precautions

Discard the product if the product or packaging is damaged or wet or if the pipette has been contaminated during unpacking or mounting.

Micromanipulation Pipettes are intended for single use only and MAY NOT BE REUSED. Reuse may cause contamination, poor embryo survival, and failed procedure.

Glass-shards may be present in spite of the 100% visual inspection, always inspect pipette before use. Do not use broken pipettes.

Caution: Federal (US) law restricts this device to sale by or on the order of a physician.

Specifications

Sterilized using dry heat	SAL 10 ⁶
Mouse Embryo Assay (1-cell) [% expanded blastocyst within 96 hours]	≥ 80
Bacterial Endotoxins (LAL assay)	≤ 1.2 EU/device

Description of ISO Symbols

STERILE I	Sterilized using dry heat.
Temperature limit	Temperature limit
Do not re-use, discard after procedure.	Do not re-use, discard after procedure.
Use by - see label.	Use by - see label.
Caution: Consult accompanying documents.	Caution: Consult accompanying documents.
Catalog number	Catalog number
Batch code	Batch code
CE Mark (Conformité Européen)	CE Mark (Conformité Européen)

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1200.08 Micromanipulation Pipettes ©Vitrolife