Transducer Covers

US

EN: Indication for use

Intended for covering the ultrasound transducer during follicle aspiration procedures.

Product Description

The Transducer Cover is a sterile, latex-free cover packed with two clips.

Storage Instructions

Store at +8 to 25°C.

Directions for use

Check that the packaging is intact and the “use by” date has not expired.

Carefully open the bag containing the transducer cover.

Apply gel over the ultrasound transducer head and insert transducer in the cover.

Use the cable clips to keep the cover in place.

Precautions

Discard the product if the packaging or product is damaged. Transducer Cover is intended for single use only and MAY NOT BE REUSED. Reuse may cause contamination and patient infection.

The product may only be used by, or under the guidance of, a person qualified in gynaecological and follicle aspiration procedures.

Caution: Federal (US) law restricts this device to sale by, or on the order of, a physician.

Specifications

Sterilized using irradiation	SAL 10 ⁶
Bacterial Endotoxins (LAL Assay)	≤ 1.0 EU/device

Description of ISO Symbols

STERILE R	Sterilized using irradiation
↕ 25°C	Temperature limit
⊘	Do not re-use, discard after procedure
📄	Use by – see label
⚠	Caution: Consult accompanying documents
REF	Catalog number
LOT	Batch code
CE 2460	CE Mark (Conformité Européen)

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26226.07 Transducer Covers © VitroLife